

AmbuMan Wireless Self-Training Module Instructions

Instructor Independent CPR Training



Step # 1

Turn the manikin on by pressing the green button on the torso

Step #2

Once the LED is a steady green connect the phone, tablet or PC to AmbuMan W wireless network

Step #3

Open web browser type in 192.168.1.1




Step #4

Click on the Self Training circle

Step #5

Enter name

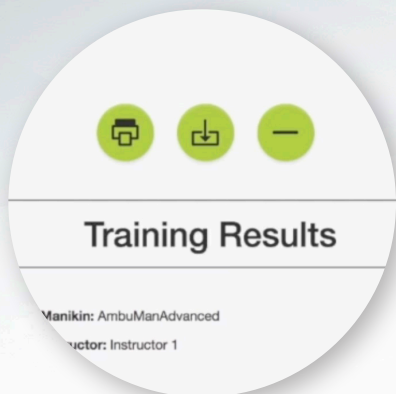

Step #6

Click Start -10 second countdown to CPR begins

Step #7

Begin CPR - When the session is complete the trainee can print, save or delete the results.